



1

Copy a Vocabulary File

- Tap on Vocab, then Choose New Vocab
- Tap on Menu, then Copy a Vocab
- Scroll to the vocabulary file you want, then tap on it
- Give the file a name and a description
- Tap on Save
- Tap on the file to open it

Pick Out a Voice

- Tap on Menu, then Settings
- Scroll down to Speech, then tap on Voice
- Use the speaker to preview a voice, tap on the voice to select it
- Voices with a cloud next to it need to be downloaded over WiFi
- Adjust Rate and Pitch if needed
- Tap on Done

If you are using bi-lingual vocabulary files, make sure to pick out a voice in that language.

2

3

Edit a Button

- Go to the page you want to edit
- Tap on Menu, then Edit Page
- Select the button to edit, then tap on Edit this Button
- Add a Button Label and Button Message
- Edit Pronunciation if needed
- Tap on Save.
- Continue editing buttons on this page, then tap on Done when finished.

4

Update Your App

- Tap on Vocab, then Choose New Vocab
- Check the bottom of the screen to for the version of your app
- Exit your app and go to the App Store
- Check for updates and run the update when ready

Back Up Your App

- Tap on Vocab, then Choose New Vocab
- Tap on Menu, then Import/Export Vocab
- Choose your method to export
- Select your file, review and change the file name if needed
- Follow steps on Quick Reference Guide to complete your export

5

6

Online Resources for Your App

- App Websites <https://aacapps.com/>
- AAC Learning Journey Website <https://www.aaclearningjourney.com>
- TouchChat App on YouTube
 - <https://www.youtube.com/touchchat1>
- LAMP Words for Life on YouTube
 - <https://www.youtube.com/c/AACandAutism>